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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

- REF.90132 - Epilastik32 Elastic elbow sleeve
- REF.90119 - Manulastik19 Elastic wrist support
- REF.90104 - Genulastik04 Mesh elastic knee brace
- REF.90137 - Malleolastik37 Elastic knit ankle support

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device. **TRADEMARKS OF MATERIALS** Velcro® is a registered trademark of Velcro Industries B.V. **SAFETY PRECAUTIONS**

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of pressure local to the compression of underlying nerves and/or blood vessels. In case of doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully. We recommend to not wear the device in the proximity of free flames or (ref.90104) strong electromagnetic fields.

ADVERTISMENTS It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist or orthopedic technician. The responsibility of the manufacturer shall remain in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In case of pain, swelling, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Code	REF.90132
Size	0 1 2 3 4 5 6
Elbow circumference taken at 13 cm from the epicondyle cm	17/19 19/21 21/23 23/25 25/27 27/29 29/31
Color	bleu

fits right and left

Code	REF.90119
Size	0 1 2 3 4 5 6
Wrist circumference cm	13/14 14/15 15/16 16/17 17/18 18/19 19/20
Color	blue

specify right or left

Code	REF.90104
Size	0 1 2 3 4 5 6 7
Thigh circumf. 14 cm above the middle of the kneecap cm	35/38 38/41 41/44 44/47 47/50 50/53 53/56 56/59
Calf circumf. 12 cm below the middle of the kneecap cm	25/28 28/31 31/34 34/37 37/40 40/43 43/46 46/49
Color	blue

fits right and left

Code	REF.90137
Size	0 1 2 3 4 5 6
Ankle circumference cm	15/17 17/19 19/21 21/23 23/25 25/27 27/29
Color	blue

fits right and left

MAINTENANCE
☒ Do not bleach ☒ No chemical cleaning
☒ Do not iron ☒ Do not tumble dry
☒ Instructions of lavage: wash at max.30°C with neutral soap.
Dry away from heat sources.)

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CONTRAINDICATIONS
Currently no known contraindications. **CHARACTERISTICS AND MATERIALS**
- Stretch fabric with different areas of compression - Anatomic design **Epilastik 32** - Silicone elements shaped for epicondylitis and medial epicondylitis and forearm press - Reinforced silicone edge, soft and traction-resisting. **Manulastik 19** - Thumb loop - Polymeric palm splint - Velcro® fastening system - Foam padding for support and traction during application - Reinforced silicone edge, soft and traction-resisting. **Genulastik 04** - Silicone patella stabiliser - Fine sprints keep the knee support firm in place - The upper edge is strengthened with silicone trim so the support feels softer and keeps its shape when being pulled on. **Malleolastik 37** - Silicone inserts shaped to fit the malleoli - The upper edge is strengthened with silicone trim so the support feels softer and keeps its shape when being pulled on.

ADAPTING / PUTTING ON THE APPLIANCE
Epilastik 32 - Slide on the elbow support with the label facing upwards; the larger part of the silicone insert should rest on the forearm - Position the elbow support so that the openings in the silicone insert are centred around the epicondyle and epitrochlea. **Manulastik 19** - Slide on the wrist support with the label facing upwards - Position the wrist support so that the thumb goes through the hole provided and the stiff splint rests against the palm of the hand **Genulastik 04** - Slide on the knee support with the label facing upwards - Arrange the knee support so that the patella is correctly positioned within the dedicated silicone stabiliser. **Malleolastik 37** - Slide on the ankle support with the label facing upwards - Arrange the ankle support so that the malleoli are correctly positioned within the dedicated silicone inserts.

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