elastop

REF. 55114
Postoperative Bandage
Post-operative support
Ceinture postopératoire
Корсет поясничный
Fascia postoperatoria

elastop

ORTHOSERVICE RO+TEN

Headquarter: ORTHOSERVICE AG
Via Milano 7 · CH-6830 Chiasso (TI) · Switzerland
Tel. 0041 91 8220088 · Fax 0041 91 8220089
info@orthoservice.com · www.orthoservice.com
Niederlassung Deutschland: Orthoservice Deutschland GmbH
Flugstraße 7 · D-76532 Baden-Baden · Deutschland
Tel. 0049 (0)7221 9719780 · Fax 0049 (0)7221 9719782
info@orthoservice.de · www.orthoservice.de
Sede italiana: RO+TEN s.r.l.
Sede legale: Via Fratelli Ruffini, 10 · I-20123 Milano (MI) · Italia
Sede operativa e amministrativa:
Via Comasina, 111 · I-20843 Verano Brianza (MB) · Italia
Tel. 0039 039 6014094 · Fax 0039 039 6014234
info@roplusten.com · www.roplusten.com
Società soggetta a Direzione e Coordinamento (art. 2497bis CC):
Orthoservice AG (CH) · 6830 Chiasso (TI) · Switzerland

9DC-F60007IN103.0517

DEUTSCH

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ENGLISH

Please read the instructions carefully and thoroughly

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares with sole responsibility that this product is a Class I medical product and has been manufactured in accordance with the basic requirements of the directives 93/42/EEC and 2007/47/EEC. These operating instructions were created in accordance with the specification of the above-mentioned directives. They aim to ensure the safe and correct use of the medical product.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

SELECTION/SIZE

Item:: REF. 55114						
Size	XS	S	M	L	XL	XXL
Chest-abdomen circumference cm	<100	100/110	110/130	130/140	140/150	150/160
Height cm	24	24	24	24	24	26

MAINTENANCE

- Do not bleach
- No chemical cleaning
- Do not iron
- Do not tumble-dry
- Washing instructions: hand wash at max. 30° C with neutral soap. Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. It is advisable to wear the product over a garment, avoiding direct contact with the skin. If in doubt about how to apply, contact an orthopaedic technician. The device should not be worn in the vicinity of open flames or strong electromagnetic fields.

WARNINGS

The product may only be used in the event of medical indication or when prescribed by a physician and must be applied by an orthopaedic technician, the competent authority figure for the application as well as for information on the safe use, according to individual needs. To ensure the effectiveness, tolerability, and correct operation, it is necessary that the device is put on with the utmost care. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. Use of the product is recommended for a single patient. With hypersensitive patients, direct contact with skin may cause redness or irritation. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician.

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