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**EN** – The descriptions and images in this document are for example and commercial purposes only. Orthoservice reserves the right to modify them according to its needs.

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## ENGLISH

### CHARACTERISTICS AND MATERIALS

- A padded fabric chest band with Velcro® that can be shortened to fit for maximum adaptability.
- An arm band and two forearm bands made from padded fabric with Velcro® which can be positioned anywhere on the chest band.

### INDICATIONS

- Glenohumeral luxation
- Distorsions of the shoulder
- Postoperative care

### CONTRAINDICATIONS

Currently no known

### APPLICATION

- 1 Chest band: wrap around the chest at elbow height, and fasten with Velcro® (fig. 1a). If too long, cut off the excess part (fig. 1b).
- 2 Arm band (orange trim): attach to the chest band using the Velcro® provided so that it is higher than the elbow (fig. 2a). Wrap it around the arm and fasten with Velcro® (fig. 2b).
- 3 First forearm band (black trim): attach to the chest band using the Velcro® provided so that it is in the middle of the forearm when the elbow is bent (fig. 3a). Wrap it around the forearm and fasten with Velcro® (fig. 3b).
- 4 Second forearm band (black trim): attach to the chest band using the Velcro® provided so that it is at hand height when the elbow is bent (fig. 4a). Wrap it around the hand and fasten with Velcro® (fig. 4b).

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