PediVis Active

DÄMPFUNGSEINLAGE - CUSHIONING INSOLE - SEMELLE AMORTISSEUSE
АМОТИЗИРУЮЩИЙ СУПИНАТОР - PLANTARE AMMORTIZZANTE

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Created with a particularly high-performing design, the insoles also support smaller joints and provide protection against impact. They are suitable for any type of activity from a simple supported walk, help to optimise posture and work the right muscle masses for the entire day.

INDICATIONS: Stabilised pronation and supination • Plantar fasciitis • Flatfoot or high arched foot • Metatarsalgia • Inflammation of the heel.

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The clear polyurethane film covering the base holds together and protects the parts of the insole. It keeps the insole firmly in place within the shoe even during very active wear.

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The anatomically designed heel support stabilises the heel area; it works with the arch support to limit pronosupination; it extends to the midfoot to also support the metatarsal bones (the shape can be altered). It contains the shock-absorbing insert.

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The shock-absorbing insert cushions any vibrations and impact force as the foot touches the ground.

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The lateral support offers a better fit for the heel and stability for the foot, even under strain.

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The strong resin shell retains its shape and guarantees proper lasting support.

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Viscoelastic memory foam for the forefoot provides an effective boost during the propulsive phase.

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The top covering is a dual-layer fabric (Lycra and Lycra Hole) that offers odour control*. The fabric undergoes a special treatment that breaks down any sweat produced by the feet and naturally neutralises the molecules that cause unpleasant odours.

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ENG. PUTTING ON THE APPLIANCE

FIRST APPLICATION

1. Remove the original insole from the patient's shoe.
2. Place the insole in the shoe, check that it is correctly positioned, and ask the patient to try it on.
3. Alterations may be made to the insole to suit individual requirements. If you wish to reshape the insoles, first preheat them in an oven at 90° for thirty seconds before heating with a heat gun from a distance of 20 - 25 cm ONLY on the side with the top covering (never heat the underside with the clear film); the insoles are now ready to be reshaped.
4. We recommend removing the insole from the shoe after wear so that it can air.

SUBSEQUENT WEAR

1. Place the insole in the shoe and check that it is correctly positioned.
2. We recommend removing the insole from the shoe after wear so that it can dry.

CONTRAINDICATIONS

Currently no known

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares with sole responsibility that this product is a Class I medical product and has been manufactured in accordance with the basic requirements of the directives 93/42/EEC and 2007/47/EEC. These operating instructions were created in accordance with the specification of the above-mentioned directives. They aim to ensure the safe and correct use of the medical product.

MAINTENANCE

Washing instructions:
Wash by hand up to 30°C with neutral soap; let it dry away from heat sources. Do not dispose of the product or any of its components into the environment. componente.

PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. It is advisable to wear the product over a garment, avoiding direct contact with the skin. If in doubt about how to apply, contact an orthopaedic technician. The device should not be worn in the vicinity of open flames.

WARNINGS

The product may only be used in the event of medical indication or when prescribed by a physician and must be applied by an orthopaedic technician, the competent authority figure for the application as well as for information on the safe use, according to individual needs. To ensure the effectiveness, tolerability, and correct operation, it is necessary that the device is put on with the utmost care. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. Use of the product is recommended for a single patient. With hypersensitive patients, direct contact with skin may cause redness or irritation. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician.

WARRANTY

This warranty is governed by the statutory provisions of the country in which the product was purchased. Where relevant that country's regulations concerning warranties and the relationship between the seller and buyer shall apply. If you believe that the terms of the warranty may be applicable, first contact the retailer where the product was purchased. Orthoservice will repair or replace all or part of the device and its accessories if defects in materials and workmanship arise during the period of six months from the date of purchase.

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