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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Knee orthosis with hinges and F-E control

DECLARATION OF CONFORMITY As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR).

TRADEMARKS OF MATERIALS Velcro® is a registered trademark of Velcro Industries B.V.

SAFETY PRECAUTIONS We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure...

WARNINGS It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient.

SELECTION/SIZE Table with columns for code, sizes (XS-XXL), and dimensions (XS-XXL).

fits right and left

MAINTENANCE

- Do not bleach, Do not iron, Do not tumble-dry, Washing instructions: hand wash at max. 30°C with neutral soap...

INDICATIONS Functional treatment of first and second sprain injuries of lateral/medial collateral ligament

CONTRAINDICATIONS Currently no known

CHARACTERISTICS AND MATERIALS

- Structure of BreathSkill: soft and breathable, 3D weave, a special woven mesh covering for a stronger, more durable design...

ADAPTING FOR THE DOCTOR/TECHNICIAN

- The knee support is supplied with the straps undone so it is ready to be adjusted. GenuSkill22 - GenuSkill26 - Slide the knee support onto the leg with the orange label at the top (fig. A)...

PUTTING ON THE APPLIANCE FOR THE PATIENT

- Undo all the straps: Undo the Velcro® fastening (fig. O), Undo the buckle (fig. P)...

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MARCHI COMMERCIALI DEI MATERIALI Velcro® è un marchio di fabbrica registrato di Velcro Industries B.V.